




## PATIENTS & BUTTONS classification and it's Store Boxes

Type	identify the PATIENTS type and behaviour	INSTITUTIONS or DEPT's.	BUTTON to Use	Recommended LIFE time from STARTING USE Date	store in BOX #	Box COLOR & Label
<b>A</b>	<b>PASSIVE Patients</b>					<b>Box # 1</b>
<b>A.1</b>	* Are Patients that <b>NEVER spin</b> de buttons * These patients <b>NEVER</b> apply high forces to the button  * <i>The patient could apply moderate forces to the button</i> * <i>But under 150 kgs traction force, maximum</i>	Hospitals Clinics Geriatrics Home Care Psychiatric	Salvafix <b>PLASTIC</b> button	* 0 to 24 months	box # 1	Reject any Button if older than <b>24 months</b>
<b>B</b>	<b>ACTIVE Patients</b>					<b>Box # 2</b>
	* Are Patients that <b>NEVER</b> apply high forces to the button  * <i>The patient could apply moderate forces to the button</i> * <i>But under 150 kgs traction force, maximum</i>	Hospitals Clinics Geriatrics Home Care Psychiatric	Salvafix <b>PLASTIC</b> button			Reject any Button if older than <b>6 months</b>
<b>B.1</b>	The patient <b>spin</b> de button <b>Time to Time</b>			* 0 to 6 months	box # 2	
<b>B.2</b>	The patient <b>spin</b> de button <b>Constantly</b>			* 0 to 3 months	box # 3	
<b>C</b>	<b>AGGRESSIVE, VIOLENT, RIOTOUS, ALTERED or other patients</b>					<b>Box # 3</b>
<b>C.1</b>	* The patients could apply moderate to high force, anytime to the button, but <b>never</b> spin it.	Urgency Psychiatric Any other	Salvafix <b>ALU</b> button	* 0 to 24 months	box # 1	Reject any Button if older than <b>3 months</b>
<b>C.2</b>	* The patient <b>spin</b> the button, time to time or constantly, applying or not moderate to high forces to the button * <i>Under 300 kgs traction force, maximum</i>			* 0 to 3 months	box # 3	
	<b>PINS</b>					<b>Box # 4</b>
	If the PIN don't show any erosion in the work area	Any	Salvafix <b>PLASTIC</b> or <b>ALU</b> Buttons	5 years	box # 4	Reject PIN's older than <b>60 months</b>

(\* ) Prior to use new Locks, you must write in each Lock the MONTH and YEAR of the **starting use date**, to achieve a good checking level.

For a better control, use a *permanent* pen to write the month and year of the starting use date.

Look for **free space to write** around the CE mark

